Shinning on your face.
Premium Skin Rejuvenation Injector

Aqua Sunny





Aquasunny

Contents

- 1. What is Aqua Sunny?
- 2. Specification
- 3. Aquasunny's mechanism
- 4. Features
 - 1) Machine
 - 2) Treatment
- 5. Solution
- 6. Indication
- 7. Treatment Procedure





What is Aqua Sunny?

- A Novel Aesthetic & Anti-aging Injection system
- Multi-Needle & Various Solution & Vacuum Injection System
- The Most Efficient & Effective Injection Treatment





Why Aqua Sunny?



Vacuum: Holds the skin for easier injection, assists safe injections for danger zones such as around the neck and the eyes







Specification


Injection Mode	Continuous/Single
Suction Level	Low, Middle, High
Syringe	2,3,5cc
Needle	9Pin, 32Gauge









Mechanism of Aqua Sunny

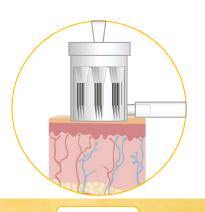



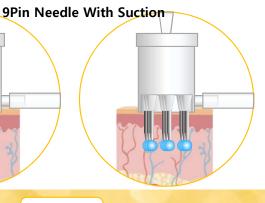

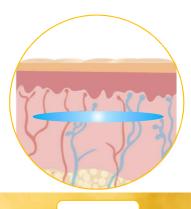

Step1

Contact to skin Vacuum applies gentle pressure on the skin and holds it to help easier injection

Step2

Operation
(Needle insertion + Injection)
The medicine is evenly injected into the right layer by injector and multineedle

Step3

End The medicine injected is spread out and the Vacuum gently releases the skin





Features "Works together to enable synergy"

- Injects equal amounts of solution into targeted depth (0~3mm)
 - -Uniformly distributed
- Fast treatment time by 9 pins
 - Less time & efforts required
- Maximize patient's satisfaction
 - -Less recovery time & pain







Solution Hydroxell (Cross Linked HA)





- Hyaluronic acid that is injected to dermis to improve wrinkles
- Effectiveness and safety is proved by clinical test
- Cross-linked HA provides longer lasting effect and safety when injected and very effective for wrinkle improvement.
- No preservatives/ Sterilized

[Indications] saggy skin, Wrinkled skin


[Main Ingredients] Hyaluronic Acid (Cross-Linked HA 15%)





Solution Hydroxell(Cross Linked HA)

<Comparison of HA>

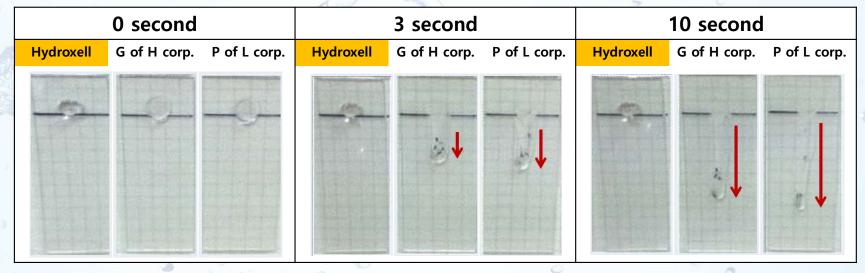







HYDROLIZED HYALURONIC ACID with OLGOPEPTIDE-1, OLGOPEPTIDE-3 AND COPPER TRIPEPTIDE-1



- High purity multi growth factor(GF) that contains skin enhancing active agents that promotes collagen synthesis and strengthens dermis to make skin look brighter
- No preservatives / Sterilized

[Indication] Aged skin, Dull skin, saggy skin

[Main Ingredients] EGF, Copper-tripeptide-1, HA, Arbutin, Bfgf, sh-Polypeptide-7, sh-Polypeptide-9, Ethyl Ascorby Ether, Defined Cell Culture Media 4







HYDROLIZED HYALURONIC ACID 95% purity with Acetyl Hexapeptide-8



- Consists with hyaluronic acid, growth factor(palmitoyl Pentapeptide-4), and Botox substitute products, Acetyl Hexapeptide-8, to be used for wrinkle around eye area, mouth, and any sunken areas
- -No preservatives/ sterilized

[Indication] Saggy skin, Wrinkled skin

[Main Ingredients] HA, Acetyl Hexapeptide-8, Palmitoyl Pent-apeptide4 Adenosine





Indication

- Dry Skin
- Fine Wrinkles & Aging Skin
- Dark and Dull Skin
- Loose Elasticity





Treatment programs

Aqua Sunny



Treatment Procedure



STEP1 Cleansing .



STEP2(30~40min) Anesthesia Cream .



STEP3(10~15min)
Treatment



STEP5Post Care





Combination Treatment



Dermaton

(RF, Vaccum, Cryo, EP)

- Solution for penetration
- -Skin Relaxation
- Improve Skin Tone



Medical Kit
(Premium Post-Laser
Treatment)



Ultraskin (HIFU)

Stimulating dermis with SAMS layer is to bring the best results such as skin tightening with rejuvenation





Youth is the flower life. Aquasunny keeps you Young & Beautiful.



